**COMPOSITION**

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Macrogols, edetic acid, carbamide peroxide

### PROPERTIES

- lubricates the root-canal and facilitates endodontic instrument penetration,
- makes shaping more efficient,
- due to its effervescent properties makes cleaning of the root canal easier.

### INDICATIONS

Facilitates root-canal preparation.

### CONTRA INDICATIONS

Hypersensitivity to any component.

### ADVERSE EVENTS

Allergic reaction.

### INSTRUCTIONS FOR USE

- Isolate the tooth to be treated, using a rubber dam ;
- Open up the pulp chamber in the usual manner ;
- Fit the applicator tip to the CANAL + syringe ;
- Place CANAL + ;
- Use for a short duration of about one minute ;
- Irrigate with a sodium hypochlorite solution to remove pulp debris and clear the entrance to the root-canal(s) ;
- Remove the liquid from the chamber by aspiration.

The way CANAL + is used will vary, depending on the anatomical features of the root-canal(s) to be prepared. Incorrect direction of reaming should be avoided in any instance. For preparation of wide and relatively straight

root-canals, CANAL + can be used starting with the first instrument number. For narrow, calcified and/or irregular shaped canals it may be necessary to assess the shape and working length and to start reaming up to a rather high number before applying CANAL +. The instruments should be pre-curved and adjusted (stopped) to facilitate the work.

Then proceed by coating the instruments with CANAL +, working in the usual way; then rinse with hypochlorite, dry and fill the canal as normal.

### Notice

Avoid using CANAL + right up to the last numbered reamer.

Rince with water between the chlorhexidine and EDTA irrigations.

Although it is highly water soluble, always make sure no trace of CANAL + remains in the canal (no more effervescence).

### WARNINGS AND PRECAUTIONS FOR USE

Danger
Causes serious eye damage.
Refer to safety data sheet (available upon request).

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