COMPOSITION

Edetate de sodium, cétramide, excipient

PROPRIÉTÉS

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Sodium edetate, cetrime, excipients

PROPERTIES

Sodium edetate dissolves the mineral content of root canals and facilitates mechanical preparation of narrow canals or canals difficult to access.

Elimination of the smear layer.

Because of its low surface tension, the product easily penetrates the narrowest canals and softens the walls.

Due to its balanced pH (6,5-7,5), it is well tolerated by the periapex.

INDICATIONS

It assists in root canal preparation by decalcifying dentin and removing the smear layer.

Facilitate locating canal orifices.

CONTRA INDICATIONS

LARGAL ULTRA is not to be used on patients having a history allergy to one of the components.

ADVERSE EVENTS

Allergic reaction.

INSTRUCTIONS FOR USE

Isolate the tooth to be treated by using a rubber dam.

Place several drops of the solution in a glass content and then place the appropriate content in the pulp cavity. Use an endodontic instrumental to make it flow along the root canal. Repeat several times. Rinse with water.

WARNINGS AND PRECAUTIONS FOR USE
In case of use of chlorhexidine, due to the risk of interaction between chlorhexidine and EDTA, a flush with distilled water is recommended in order to remove traces of chlorhexidine inside the root canals before irrigation with this solution.

Causes skin irritation.
Causes serious eye irritation.
Refer to safety data sheet (available upon request).

STORAGE
Close quickly and correctly the bottle after use.

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