COMPOSITION

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EN

COMPOSITION

3% sodium hypochlorige
Excipients q.s ad 100 g
(Sodium chloride, sodium carbonate, sodium hydroxide solution, sodium edetate, purified water).

PROPERTIES

The root canal preparation, essential procedure for the success of the obturation, needs the use of irrigant solutions combined with a chemo-mechanical reaming. PARCAN is a 3% sodium hypochlorige stabilized and buffered solution, ready for use that meets the requirements of the expected effects of an irrigation solution, i.e.: - chemical effects by dissolution of the organic matrix of the dentin, allowing to flare the canal easily; - secondary bactericidal action of sodium hypochlorige on root canal flora.

INDICATIONS

Canal irrigation for root canal preparation.

CONTRA INDICATIONS

Hypersensitivity to sodium hypochlorige.
Immature teeth or teeth with root resorption.

ADVERSE EVENTS

- Allergic reactions
- Accidental exposure of tissue with sodium hypochlorige can lead to local swelling, face oedema, mouth oedema, hematoma, hypoesthesia and tissue necrosis.

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INSTRUCTIONS FOR USE

Pump the PARCAN solution with a syringe. Attach an endodontic irrigation needle to the syringe. Insert the needle in the root canal and dispense the sodium hypochlorige solution.

WARNINGS AND PRECAUTIONS FOR USE

- Plastic bib to protect patient's clothing
- Provision of protective eye-wear for both patient and operator
- Root canal irrigation, as with all root canal therapies, should be conducted using a dam. If for technical reasons, the dam cannot be inserted, the region should be isolated using a cotton roll and local aspiration to avoid swallowing of the product by the patient.
- So as to eliminate any risks of injecting the product beyond the apex, it is strongly recommended:
• to check the apical closure;
• to avoid using of the product under pressure.
- To avoid injection of hypochlorige in the periapical tissues, it is advisable to use a special needle with lateral openings, to insure that the injected liquid does not penetrate the periapical region.
- Do not swallow.
- When irrigation associates sodium hypochlorige and chlorhexidine, rinse with water between the two irrigants to avoid any interaction and precipitate formation.

STORAGE

Keep away from light at temperature ≤ 25°C.

PRESENTATION

- Perfusion-type bottle of 250 ml solution ready for use.
- 3 x 120 ml bottle of ready to use solution.

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EIGENSCHAFTEN

- Babador plástico para proteger as roupas do paciente
- Fornecimento de proteção ocular tanto para o paciente como para o operador
- Root canal irrigation, as with all root canal therapies, should be conducted using a dam. If for technical reasons, the dam cannot be inserted, the region should be isolated using a cotton roll and local aspiration to avoid swallowing of the product by the patient.
- So as to eliminate any risks of injecting the product beyond the apex, it is strongly recommended:
• to check the apical closure;
• to avoid using of the product under pressure.
- To avoid injection of hypochlorige in the periapical tissues, it is advisable to use a special needle with lateral openings, to insure that the injected liquid does not penetrate the periapical region.
- Do not swallow.
- When irrigation associates sodium hypochlorige and chlorhexidine, rinse with water between the two irrigants to avoid any interaction and precipitate formation.

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