Septodont Studies No. 15 - May 2017 Septodont Studies No. 15 - May 2017

Dentinal bridge formation with Biodentine™

Dr. A. Itri



Tricalcium silicate as cavity base

Dr. A. Segovia Ramírez Dr. D. Álvarez Rodríguez



Biodentine™: indirect pulp cap

Dr. S. Patel Dr. L. Vincer



BIOROOT™ RCS

A reliable bioceramic material for root canal obturation Dr. Jenner O. Argueta







Editorial



Since its foundation Septodont has developed, manufactured and distributed a wide range of high quality products for dental professionals.

Septodont recently innovated in the field of gingival preparation, composites and dentine care with the introduction of Racegel, the N'Durance[®] line and Biodentine[™], which are appreciated by clinicians around the globe.

Septodont created the "Septodont Case Studies Collection" - a series of case reports - in 2012 to share with you their experience and the benefits of using these innovations in daily practice.

Over the past 5 years, authors from more than 15 countries have generously contributed to the success of our magazine that is now distributed on the 5 continents.

Each new issue of the Case Studies Collection is the opportunity to discover new clinical challenges and their treatment solutions.

This 15th issue features three Biodentine[™] cases and a new case report on the most recent Septodont Innovation : BioRoot[™] RCS.

- BioRoot[™] RCS is the new paradigm for endodontic obturations. Its outstanding sealing properties combined with antimicrobial and bioactive properties allow to get a high seal of the endodontium without having to use complex warm gutta techniques.
- Biodentine™, the first biocompatible and bioactive dentin replacement material. Biodentine™ uniqueness not only lies in its innovative bioactive and "pulp-protective" chemistry, but also in its universal application, both in the crown and in the root.


Content



BioRoot[™] RCS, a reliable bioceramic material for root canal obturation

04

Dr. Jenner O. Argueta

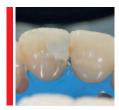

Dentinal bridge formation: clinical results after Biodentine[™] removal

10


Dr. Angelo Itri



Tricalcium silicate as cavity base: clinical case report

15

Dr. Alberto Segovia Ramírez & Dr. Deicy Álvarez Rodríguez

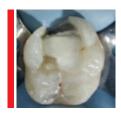

Single Visit Indirect Pulp Cap using Biodentine $^{\text{\tiny TM}}$

21

Dr. Shanon Patel & Dr. Louise Vincer

BioRoot™ RCS, a reliable bioceramic material for root canal obturation Jenner O. Argueta D.D.S. - M.Sc.

Introduction

During the treatment of root canals it is practically impossible to obtain an environment completely free of bacteria (Markus Haapasalo, Shen, Qian, & Gao, 2010); considering this fact, the root obturation procedure must contain the remaining microorganisms, keeping them deprived of nutrients and of an environment capable of reactivating their metabolism and growth (Simon, 2016; Siqueira, Araujo, & García, 1997).

The majority of modern obturation techniques involve the use of gutta-percha combined with sealing cement; the latter is used with a view to filling in the interface between the root dentin and the gutta-percha. Cement fluidity is an important factor in ensuring that it will reach the areas of the canal that cannot be accessed with root-shaping instruments, but are receptive to chemical disinfection processes applied by means of the various irrigation techniques (Siqueira, Rocas, Favieri, & Lima, 2000). It is advisable to use a minimal amount of sealing cement in proportion to the amount of guttapercha used, when resin-, zinc oxide-eugenol-, or calcium hydroxide-based cements are to be used, since the use of substantial amounts of cement generates the possibility of degradation

and leakage, which may lead to bacterial recontamination, and thus causing over time the failure of the endodontic treatment (Simon, 2016).

Of the obturation techniques discussed in the literature, single-cone obturation is one of the most sensitive to post-operative leakage since the gutta-percha cones used with the instrumentation system are not perfectly compatible with the final shape of the root canal (Schäfer, Köster, & Bürklein, 2013).

Due to the variability of gutta-percha cones and the irregularities specific to root canal systems, the sealing cement used must be physically stable, must provide good apical sealing, and must have the ability to set in the presence of the moisture present in dentin and periradicular tissues. Single-cone obturation is one of the simplest and quickest methods to use, but is very questionable if applied with non-bioceramic cements, since the presence of large amounts of sealant in the obturation may cause leakage problems over time (Simon, 2016).

Bioceramic cements are an interesting option for the use of the single-cone technique; their physical characteristics render them capable of providing a stable three-dimensional seal in the necessary time frame (Daculsi, Laboux, Malard, & Weiss, 2003), all without the need for compaction procedures, whether warm or cold. These materials are able to set in humid environments; this point has major relevance considering the fact that dentin has a moisture content of approximately 20%, and that work in moisture-saturated environments is a constant in the dental profession (K. Koch, Brave, & Nasseh, 2010).

Bioceramic cements are divided into three basic groups.

- 1: Bioinert high strength cements;
- 2: Bioactive cements that form chemical bonds with mineralized tissue; and
- 3: Biodegradable materials that integrate actively with the body's metabolic processes (K. Koch & Brave, 2009).

Due to their high stability and sealing properties, bioceramic cements can be used in combination with gutta-percha as part of a single-cone technique, or directly inside the root canal to seal their entire length. Though bioceramic cement may function as an obturation material, it is advisable that a gutta-percha cone be used to

convey it to the inside of the canal and hold it in position at working length or one millimeter short, to leave a route for re-treatment, if necessary in the future. This last procedure would be a real challenge for the operator if no access route were available for re-treatment. The single-cone obturation technique can be used safely in combination with bioceramic cements, due to their previously mentioned physical and dimensional stability, good sealing properties, antibacterial potential, biocompatibility, and bioactivity capable of stimulating periapical tissue repair (Trope & Debelian, 2014).

BioRoot™ RCS is a relatively new bioceramic cement based on tri-calcium silicate, zirconium oxide as a biocompatible radio-opacifying material and a hydrophilic polymer to improve its adhesion properties; the liquid mostly contains water with calcium chloride as a setting modifier (Nakov et al., 2015). The working time is approximately 15 minutes and the total setting time is 4 hours within the root canal (Simon, 2016). Next, we present a clinical case performed using BioRoot™ RCS as a root filling cement.

Clinical Case

A 45-year-old patient reported the loss of a cervical restoration in tooth 12, the caries present in the area was removed to verify the extent of the lesion (*Fig. 1*). Thermal sensitivity tests and periapical radiographs were performed (*Fig. 2*). In consideration of all the signs and symptoms



Fig. 1: Cervical cavity after elimination of carious lesion; the root canal was exposed during mechanical tissue removal.



Fig. 2: Initial x-ray - note the periapical areas shown.

present, a diagnosis of pulpal necrosis with asymptomatic apical periodontitis was made. The root canal treatment was performed in one session, first restoring the cervical cavity with resin-reinforced glass ionomer (Fig. 3). To prevent the restoration material from causing an obstruction within the duct, a No. 20 k-file was placed in the cervical radicular area (Fig. 4). With the operating environment properly prepared to achieve good isolation, the canals were permeabilized to a #15.02 hand file with the pulp chamber filled with EDTA 17% Gel (MD-Chelcream) after said chamber was first disinfected with sodium hypochlorite 5.25%. The canal system configuration was determined to be Vertucci Type IV (Altunsoy, Nur, Aglarci, Colak, & Gungor, 2014).

Mechanized instrumentation was performed with TF Adaptive files using adaptive motion, after determining the working length using the Rootor electronic apex locator. During the procedure, a #25.06 instrument was separated in the apical region of the palatal canal (*Fig. 5*); the latter was bypassed and then obturated using the hydraulic condensation technique (J. Koch & Brave, 2012) using BioRoot™ RCS as a sealing cement.

The last x-ray shows cement puffs in both roots (Fig. 6 and 7). The patient was asymptomatic during the postoperative period. At the reevaluation appointments, healing was observed to be ongoing at 5 months (fig. 8) and complete healing was observed at the final re-evaluation, performed 9 months after the initial procedure was completed (fig. 9).



Fig. 3: Photograph of the restoration in the cervical area prior to the root canal treatment.



Fig. 4: Placement of a K-file in the buccal canal prior to obturation placement in the cervical region; the instrument was put in place to keep the pathway permeable.



Fig. 5: Separated instrument in the palatal canal; this was bypassed prior to the final obturation.



Fig. 6: Final x-ray taken at an orthoradial angle; cement puffs can be observed in the periapical regions.



Fig. 7: Final mesial x-ray; the separated instrument included in the final obturation can be



Fig. 8: Re-evaluation at 5 months, the periapical areas are seen to be in the process of healing; part of the cement has been resorbed by the organism.



Fig. 9: Re-evaluation at 9 months, apical lesions have completely healed. There has been partial resorption of the apically extruded cement.

Discussion

Root canal treatment is performed in view of avoiding periradicular lesions or otherwise of promoting an adequate environment for the body to be able to heal the existing lesion or pathology (Peters, 2004); the use of cements to seal the interface between tooth and guttapercha is of crucial importance in achieving the objective mentioned earlier. Bioceramic cements such as BioRoot™ RCS are made from a combination of silicate and calcium phosphate; the bonding of these components provides physical and biological properties such as: Alkaline PH, anti-bacterial activity and bio-compatibility (Candeiro, Correia, Duarte, Ribeiro-Siqueira, & Gavini, 2012). Another advantage of this type of material is its ability to form hydroxyapatite and even bring about bonding between dentin and root canal obturation material during the setting process (Loushine, Bryan, & W., 2011). The latter characteristic is of high importance in repair processes, since, as can be observed in the present case, the release of ions linked to mineralization processes may promote the complete and relatively rapid healing of the periapical lesions. The biocompatibility of the cement is also apparent in this case: despite

the puffs that were produced because of the obturation technique used, the patient remained completely asymptomatic and symptom-free. One of the most common techniques of root obturation using bioceramic cements is to place small amounts of cement into the canal, bringing it up to the proper length using paper tips; the use of paper tips also has the advantage of eliminating the excess moisture present in the cement and limiting the apical pressure exerted. After the cement has been placed as desired, a gutta-percha cone is inserted to constitute the central part of the obturation (M. Haapasalo, Parhar, Huang, Way, & James, 2015).

BioRoot™ RCS has been increasingly popular since its introduction, and has become one of the materials of choice in cases of open apices and extensive periapical lesions; its popularity is due in large measure to its excellent biocompatibility, remarkable sealing properties, hydrophilicity, and its capacity to promote both healing and tissue mineralization (Simon, 2016) (J. D. Koch & Brave, 2012). Together these properties make BioRoot™ RCS a very interesting option when choosing an obturation technique.



Author:
Jenner O. Argueta Zepeda
Dental surgeon
Master in Endodontics
President, Guatemalan Endodontics Academy.
Professor of Endodontics at Mariano Gálvez University, Guatemala.
International Lecturer.

Researcher at the Guatemalan National Council for Science and Technology (CONCYT).

Member of the American Association of Endodontists.

References

- Altunsoy, M., Nur, B. G., Aglarci, O. S., Colak, M., & Gungor, E. (2014). A cone-beam computed tomography study of root canal morphology of maxillary and mandibular premolars in a Turkish population. Acta Odontol Scand, 72(8), 701-706. doi:10.3109/00016357.2014.898091
- Candeiro, G. T., Correia, F. C., Duarte, M. A., Ribeiro-Siqueira, D. C., & Gavini, G. (2012). Evaluation of radiopacity, pH, release of calcium ions, and flow of a bioceramic root canal sealer. J Endod, 38(6), 842-845. doi:10.1016/j.joen.2012.02.029
- Daculsi, G., Laboux, O., Malard, O., & Weiss, P. (2003). currrent state of the art of biphasic calcium phosphate bioceramics. Journal of Materials Science: Materials in Medicine, 195-200.
- Haapasalo, M., Parhar, M., Huang, X., Way, X., & James, L. (2015). Clinical use of bioceramic materials. Endodontic Topics, 32, 97-117.
- Haapasalo, M., Shen, Y., Qian, W., & Gao, Y. (2010). Irrigation in Endodontics. Dental Clinics, 54.
- Koch, J. D., & Brave, D. (2012). Bioceramics, Part 1: The Clinician's Viewpoint. Dent Today.
- Koch, K., & Brave, D. (2009). Endosequence: melding endodontics with restorative dentistry, part 3. Dent Today, 28(3).
- Koch, K., Brave, G., & Nasseh, A. A. (2010). Bioceramic technology, Closing the endo-restorative circle, part 2. Dent Today, 29(3).
- Loushine, B. A., Bryan, T. E., & W., L. S. (2011). Setting properties and cytotoxicity evaluation of a premixed bioceramic root canal sealer. Journal of Endodontics, 37(7).
- Nakov, D., Uzunoglu, E., Ardila-Osorio, H., Baudry, A., Richard, G., & Kellermann, O. (2015). Bioactivity of BioRoot™ RCS, a root canal sealer, via A4 mouse pulpal stem cells in vitro. Dental Materials, 31(11).
- Peters, O. A. (2004). Current challenges and concepts in the preparation of root canal systems: a review. J Endod, 30(8), 559-567.
- Schäfer, E., Köster, M., & Bürklein, S. (2013). Percentage of gutta-percha-filled areas in canals instrumented with nickeltitanium systems and obturated with matching single cones. Journal of Endodontics, 39(7).
- Simon, S. F., T. (2016). BioRoot RCS a new biomaterial for root canal filling. Septodont Case Studies Collection, 13, 4-10.
- Siqueira, J. F., Jr., Araujo, M. C. P., & García, P. F. (1997). Histological evaluation of the effectivness of five instrumentation techniques for cleaning the apical third of root canals. Journal of Endodontics, 23(8).
- Siqueira, J. F., Jr., Rocas, I. N., Favieri, A., & Lima, K. C. (2000). Chemomechanical reduction of the bacterial population in the root canal after instrumentation and irrigation with 1%, 2.5%, and 5.25% sodium hypochlorite. J Endod, 26(6), 331-334. doi:10.1097/00004770-200006000-00006
- Trope, M., & Debelian, G. (2014). Bioceramic Technology in Endodontics. Inside Dentistry.





We know that failure of a root canal treatment can have severe consequences for you and your patients.

With BioRoot™ RCS, move to a new generation of mineral obturation offering you an innovative combination of features:

- High Seal
- Antimicrobial properties
- Promotes peri-apical healing
- Easy obturations and follow-up

BioRoot™ RCS. Succeed.





Dentinal bridge formation: clinical results after Biodentine™ removal

Angelo ITRI, D.D.S.

Introduction

Vital Pulp Therapy (VPT) is a biologic and conservative treatment modality to preserve the vitality and function of the coronal or remaining radicular pulp tissue in vital permanent teeth (Akhlaghi et al. 2015). In this conservative therapy group there are: indirect pulp capping, direct pulp capping, miniature pulpotomy and full pulpotomy (Ghoddusi et al. 2014) . When one of these therapy modalities are performed, the most important biological target is dentinal bridge formation. Dentinal Bridge is a new reaction tissue that preserves pulp vitality and protects it from abnormal and continuous physical stimuli (cold, hot, percussion); it is due to a series of inflammatory responses leading to the formation

of a hard-calcified dentin. This tissue was observed for the first time after calcium hydroxide application on a pulp exposure; owing to a high pH (12,5) calcium hydroxide causes liquefaction necrosis of the superficial pulp, removing up to 1,5 mm of the inflamed tissue. This is followed by neutralization of toxicity in the deep layers with coagulative necrosis. Clinically, dentinal bridge formation is valued by Rx analysis where the pulp chamber is reduced after 3-6 months from VPT therapy.

The aim of this article is to demonstrate a real dentinal bridge formation after complete $Biodentine^{T}$ removal.

Clinical Case

A 29 year-old female patient with no systemic disease came to my office to restore 2.1 element where was a carious process on the mesial side (Fig. 1).

No spontaneous pain. Her dentist had made a conservative therapy two times in 4 months where the direct restoration had an incorrect adhesion. After a clinical analysis of Rx, cold, electric and percussion tests, a carious process

close to the pulpal horn was detected (Fig.2 - 3). After obtaining the patient's consent, local anaesthesia was performed (Articaine HCL 4% and 1:200000 adrenaline, Septanest, Septodont), the carious dentine was completely excavated and the pulp was exposed (fig 4-5-6-7).

A miniature pulpotomy by low speed handpiece was performed and Biodentine[™] (Septodont) (Fig.8) was placed.



Fig. 1: Preoperative clinical photograph of carious process on 2.1 and 1.1.



Fig. 2: Preoperative Rx of carious process on 2.1 and 1.1; the pulpal horn was very close to the carious process.

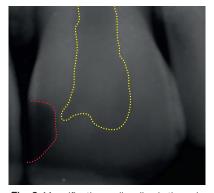


Fig. 3: Magnification: yellow line is the pulp chamber contour; red line is the cavity contour. It shows that the carious process is 1mm close to the pulpal horn.



Fig. 4: Preoperative clinical photograph. The carious process looks very small and simple to achieve restorative success.



Fig. 5: A steel matrix protect element 1.1 during carious excavation. Presence of deep carious process under the initial cavity.



Fig. 6: Pulpal exposure (palatal view).

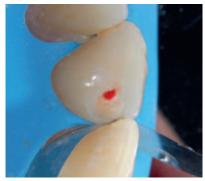


Fig. 7: Pulpal exposure (occlusal view).



Fig. 8: Biodentine™ in situ during VPT procedure.

To improve hardness resistance a thin Equia Coat (GC) layer was applied on Biodentine[™] restoration (*Fig.9*). Cold, Electric and Percussion Analysis were performed at 7,14,30,60,90, 180 and 360 days (*table 1*). After 360 days second restaurative step was performed (*fig.10*).

After obtaining the patient's consent, local anaesthesia was performed (Articaine HCL 4% and 1:200000 adrenaline, Septanest, Septodont), and the Biodentine[™] restoration was completely

removed to observe dentinal bridge type formation (*Fig.11*). In this case a complete Biodentine[™] removal was performed only to value real dentinal bridge formation; clinically this procedure is absolutely unnecessary to achieve clinical success. A composite direct restoration (ASTERIA) was performed on 2.1 and 1.1 elements (*fig.12a*, 12b, 12c).

No apical suffering signs were detected (Fig. 13) and clinical tests were normal (Table 1).



Fig. 9: Biodentine™ after equia coat layer application.

tooth 21	electric	cold	percussion
pre op	17	+	-
7 days	17	++	-
14 days	11	++	-
30 days	18	++	-
60 days	10	++	-
90 days	10	+	-
180 days	15	+	-
360 days	15	+	-
Table 1: clinical tests			



Fig. 10: Biodentine™ after 13 months (palatal view).



Fig. 11: Dentinal Bridge Formation after Biodentine™ removal.

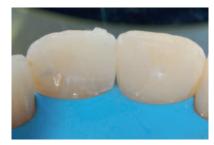


Fig. 12c: Direct composite restoration of 2.1 and 1.1.



Fig. 12: Dentinal Bridge Formation after Biodentine™ removal (Fig. 11 magnification).





Fig. 12a, 12b: Direct composite restoration of 2.1 and 1.1.



Fig. 13: rx analysis post op.

Discussion

Clinical practice success on VPT procedure is made when no spontaneous pain is present. Dausch et al. (1952) demonstrated that, after a pulp amputation and calxyl application, clinical success was 98% while histologically only 76%. Dentinal bridge formation is an important Rx success parameter. In recent articles, dentine bridge formation was evaluated. Nowicka et al. (2015) performed on 44 third molars without carious process a voluntary pulp exposure and analysed dentinal bridge thickness and density after application of four types of materials. The results show that Biodentine™ and MTA resulted in the formation of bridges with a significantly higher average volume compared with Single Bond Universal.

Only The CBcT offers a correct volumetric Rx analysis of dentinal bridges, but in common practice a 2D evaluation had never been made. Dentinal Bridge is made of colliquative and/or liquefaction necrosis processes. This events create a hard tissue layer that protect the residual pulp tissue by chemical-physical stimulus. When a VPT is performed and a restoration is made, an empty space is present between a pulp capping material and a dentinal bridge caused by superficial pulp destruction (Franz et al. 1984). Dentine fragments, which are displaced into the pulp during cavity preparation, are acting as initial loci for mineralization or pulp stone formation (Mjor et al. 1991).

Angiogenesis is essential for pulp wound-healing process because blood vessels play an important role in nutrition and oxygen supply, as a conduct for transport of metabolic waste, pulp homoeostasis and metabolism, and stem/progenitor cell

migration (Nakashima et al. 2005). A correct material choice in VPT is very important to improve cells vitality. In recent research has been demonstrated that Biodentine™ caused a cytotoxic effect similar to MTA, suggesting that it may be considered an alternative in pulp-capping treatment (Poggio et al. 2015).

Dentinal Bridge formation time is 90 days (Horsted et al. 1981). In my clinical practice, in Rx analysis the dentinal bridge presence is clear after 180-360 days. This observation highlights that a dentinal bridge is one of the parameters of a VPT follow up, but the clinical waiting period observation is about 1 year.

Matsuo et al. (1996) demonstrated that dentinal bridge formation was not clearly detected in Rx analysis and in his study, he did not include reparative dentine formation as a criterion for success and observational period was more variable (3 to 24 months).

Hyper-sensitivity after VPT procedure has never been present in a period of about 30-40 days. This characteristic is due to presence of the inflammation that is a prerequisite for tissue healing and pulp regeneration (Goldberg et al. 2015).

Conclusion

VPT success in clinical practice depends on many factors. A correct material choice is helpful to achieve pulp vitality and dentinal bridge formation after a pulp capping procedure.



Author: Dr. A. Itri

Angelo Itri graduated in 2009 and works as a general dentist in his office. He won 2010 Duillo research award in Genoa. In 2012 He was a supervisor at Bioengineering University of Genoa. Dr. Itri is actively involved in research since 2006, focusing on dental implants, dental materials and prosthodontics. He participates in national and international dental conferences.

References

- Hersted P, El Attar K, Langeland K. Capping of monkey pulps with Dycal and a Ca-eugenol cement. Oral Surg Oral Med Oral Pathol. 1981 Nov;52(5):531-53.
- Daush H, Sauerwein E. The effect of calxyl on the pulp tissue. Deutshe Zahnarztliche zeitschrift. 1952 Dec.
- Nowicka A, Wilk G, Lipski M, Kołecki J, Buczkowska-Radlińska J. Tomographic Evaluation of Reparative Dentin Formation after Direct Pulp Capping with Ca(OH)2, MTA, Biodentine™, and Dentin Bonding System in Human Teeth. J Endod. 2015 Aug;41(8):1234-40. doi: 10.1016/j.joen.2015.03.017. Epub 2015 May 29.
- Franz FE, Holz J, Baume LJ. Ultrastructure (SEM) of dentine bridging in the human dental pulp. J Biol Buccale. 1984 Sep;12(3):239-46.
- Akhlaghi N, Khademi A. Outcomes of vital pulp therapy in permanent teeth with different medicaments based on review of the literature. Dent Res J (Isfahan). 2015 Sep-Oct;12(5):406-17.
- Ghoddusi J, Forghani M, Parisay I. New approaches in vital pulp therapy in permanent teeth. Iran Endod J. 2014 Winter;9(1):15-22. Epub 2013 Dec 24.
- Goldberg M, Njeh A, Uzunoglu E. Is Pulp Inflammation a Prerequisite for Pulp Healing and Regeneration? Mediators Inflamm. 2015;2015:347649. doi: 10.1155/2015/347649. Epub 2015 Oct 11.
- Mjör IA, Dahl E, Cox CF. Healing of pulp exposures: an ultrastructural study. J Oral Pathol Med. 1991 Nov; 20(10):496-501.
- Nakashima M, Akamine A. The application of tissue engineering to regeneration of pulp and dentin in endodontics. J Endod. 2005 Oct;31(10):711-8.
- Poggio C, Ceci M, Dagna A, Beltrami R, Colombo M, Chiesa M. In vitro cytotoxicity evaluation of different pulp capping materials: a comparative study. Arh Hig Rada Toksikol. 2015 Sep 1;66(3):181-8. doi: 10.1515/aiht-2015-66-2589.





Alberto Segovia Ramírez, D.D.S. Deicy Álvarez Rodríguez, D.D.S.

Abstract

Dental cements are used as linings for the protection of the pulp dentine complex. In 1970, Wilson and Kent developed the glass ionomer in order to restore class III and V cavities. In 2009 Septodont developed a material to replace dentine whose active ingredient is pure tricalcium silicate. It has characteristics such as biocompatibility, antibacterial action, mechanical strength,

short setting time, easy handling, repair of pulp dentine, marginal adaptation and sealing. One of the indications of Biodentine^{$^{\text{TM}}$} is base for pulp protection and the regeneration of secondary dentine. A case report is presented in which tricalcium silicate is used as a base in tooth number 48, restored with resin.

Introduction

In dentistry there is a wide range of dental materials that have been modified over time in line with clinical demands, the objective being to achieve uniform biological, physical and chemical characteristics of teeth.

In order to understand the mechanics of the materials and choose the best ones, depending on the circumstances and needs of each patient, it is important to have detailed knowledge of his/her histology and dental physiology.

The histogenetic classification of dentine is split into three types:

- 1. Primary dentine
- 2. Secondary dentine
- 3. Tertiary dentine (1)

Some dental cements are able to induce tertiary

dentine or repair, such as calcium hydroxide, glass ionomer and mineral trioxide aggregate (MTA).

A new dental material now exists whose characteristics induce secondary dentine: pure tricalcium silicate marketed by the company Septodont under the name of Biodentine $^{\text{TM}}$.

The indications of dental cements are:

- Cementing agents
- Pulp protection agents (2)

For pulp protection, the following have to be taken into account:

- 1. Chemical protection
- 2. Electrical protection
- 3. Thermal protection
- 4. Pulp medication
- 5. Mechanical protection (3)

BASE

This is a cement applied as an insulating layer, acting as medication. It is inserted in the deepest part of the preparation to protect pulp tissue from thermal or chemical aggressions. Another of its functions is to protect pulp capping against different types of aggression (2).

This one has a thickness of 1-2 mm, being thicker than most liners. Its function is thermal insulation and it acts as a substitute for dentine, creating mechanical support for the restoration due to the distribution of tension on the dentine surface, achieving dental rigidity as well as being biocompatible. (3) (4) (5) (6) (7) (8)

At present, the two materials most used as bases are: glass ionomer and pure tricalcium silicate.

GLASS IONOMER

Developed in 1970 by Wilson and Kent.

The characteristics of glass ionomers are:

- Biocompatibility.
- Chemical and/or physical-chemical adhesion to enamel, dentine and cement.
- Fluoride release.
- Compressive strength: 90-230 MPa.
- Low microfiltration due to the thickness of the film.

An ionomer in powder form contains silicate glass powder, lanthanum, strontium, barium and zinc oxide.

The liquid of the ionomer is an aqueous solution consisting of water and polyalkenoic or tricarboxylic acid;

- 1. tartaric acid
- 2. itaconic acid
- 3. maleic acid (9) (10)

The classification of the glass ionomers according to their indication is as follows:

- Type I: crowns, bridges and the cementing of orthodontic bands (luting cement)
- Type II a: Aesthetics of restorative cements
- Type II B: Reinforcement of restorative cements
- Type III: cement liners, bases

The use of glass ionomer is a solution for pulp protection due to its characteristics. The disadvantage is its solubility. (9)

PURE TRICALCUIM SILICATE

In 2009 Septodont launched Biodentine[™] on the market as a "dentine replacement".

The properties of pure tricalcium silicate are the following:

- Repair of the pulp dentine complex.
- Short setting time.
- Antibacterial properties.
- · Biocompatibility.
- · Bond strength.
- Easy to handle.
- Marginal adaptation and sealing capacity. (11)(12)(13)

The material comes in capsules (powder) and pipettes (liquid). This presentation makes it easier to handle. The capsules contain a hydrophilic powder with the following composition: tricalcium silicate (3CaO.SiO₂) [primary nucleus], dicalcium silicate (2CaO.SiO₂) [second nucleus], and also calcium carbonate (CaCO3) and zirconium dioxide (ZrO₂). The pipette contains calcium chloride dihydrate (CaCl₂.2H₂O), which speeds up setting time and improves the physical properties of the material. The pipette also contains a hydrosoluble polymer, introduced to reduce the viscosity of the material. This increases

the material's resistance over a short period as well as facilitating its handling. The pipette also contains water (H_2O). (11) (12) (13)

Pure tricalcium silicate is marketed as Biodentine[™] by Septodont.

Instructions for use: put 5 drops from the pipette in the capsule and close the capsule. Put it in the triturator for 30 seconds, then open the capsule to work with the material, with a setting time of 6-12 minutes. (11)

Its properties, biocompatible with dentine and the periodontal ligament, increase its initial mechanical resistance of 131.5 MPa. During the month this figure increases and stabilizes, reaching a mechanical resistance similar to that of dentine (297MPa). Other calcium-based cements do not achieve this. (12) (13)

According to studies made on tricalcium silicate, this cement is not cytotoxic, mutagenic, sensitizing or irritating. (Zhou et al, 2013; Rodriguez

et al, 2014). Its alkaline pH is 12.3, i.e. it has an antibacterial function (12)

Garrido M. et al. presented a biocompatibility study comparing Biodentine[™], MTA, zinc oxide and eugenol, in subcutaneous tissue. The biocompatibility of pure tricalcium silicate was demonstrated with a slight to insignificant inflammation over 14 days. (14)

The indications for tricalcium silicate are:

- Base in deep cavities
- Direct restoration
- Indirect restoration
- Direct pulp lining after exposure due to caries
- Direct pulp capping following a dental trauma
- Repair of perforation in root canals or pulp chamber floor
- Retrograde endodontic surgery
- Pulpotomy in temporary molars
- Apexification.

Report of a clinical case

Male patient, 36 years of age, comes for consultation (revision). Suffers discomfort when eating sweet foods in a localized and spontaneous manner.

In the physical examination, pigmentation in the vestibular groove of tooth 48 is observed. Vitality tests positive without pulp disorder in teeth 47 and 48.

In the X-ray scan a two-dimensional image with a radiolucent lesion is observed, apparently around the pulp chamber. Two well-defined roots are observed without any periapical disease in tooth 48 (Fig. 1).

Diagnosis: caries in the vestibular sulcus of tooth 48.

Treatment: resin in tooth 48.

In the operative procedure the patient was anaesthetized with 2% Mepivacaine using the trunk technique. Three cartridges were infiltrated, complete insulation performed, followed by a removal of the caries with a 330 bur. Once the



Fig. 1: (9 April 15) Initial X-ray.

cavity was prepared, Biodentine[™] was inserted (Figs. 2, 3, 4).

Five drops from the pipette were introduced into the capsule (BiodentineTM). (Figs. 5, 6, 7) The capsule was placed in the triturator for 30 seconds and the tricalcium silicate inserted into the cavity. (Figs. 8, 9). We filled with resin

(IPS Empress) A2 dentine and A2 enamel in tooth 48. (Figs. 10, 11, 12) and took the final periapical X-ray and bitewing with the parallel plane technique (Fig. 13).

Control X-rays were taken at 1 year and 17 days after the treatment (*Figs. 14, 15*) and vitality tests were performed, with positive results.



Fig. 2: Removal of caries - tooth 48.



Fig. 3: Removal of caries -tooth 48.



Fig. 4: Removal of caries - tooth 48.



Fig. 5: The pipette is opened (Biodentine TM).

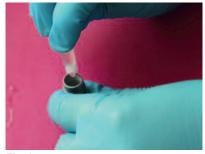


Fig. 6: Five drops from the pipette are placed in the capsule.



Fig. 7: Capsule closed for later mixing in the triturator.



Fig. 8: Transfer of the tricalcium silicate to the deep cavity.



Fig. 9: Placement of the tricalcium silicate in the deep cavity as a base.



Fig. 10: A2 resin (IPS Empress) on the base.



Fig. 11: A2 resin (IPS Empress) on the base.



Fig. 12: Final restoration.



Fig. 13: (9 April 15). Final X-ray.



Fig. 14: (26 April 16). Control X-ray at 12 months and 17 days.



Fig. 15: (26 April 16). Control X-ray at 12 months and 17 days.

Discussion

It is very important to know the biological, mechanical, physical and chemical characteristics of dental materials in order to use them correctly and obtain benefits in the medium and long terms.

Previously, if one had a deep cavity in operative restorations the Sandwich technique was used, consisting of indirect pulp capping and a liner to protect the pulp cap. (10)

Nowadays, one of the best pulp capping and lining materials is tricalcium silicate (Biodentine™), with verified positive results. This facilitates a single-step clinical approach - indirect pulp cap and liner at the same time -

that makes the operative process easier and reduces the time involved.

Conclusion

At 12 months and 17 days the patient is asymptomatic. There are no clinical signs nor symptoms of pulpal necrosis and the tooth is vital. We observed the restorative material by X-ray (radiopaque). The success obtained in this clinical

case is due to knowledge and selection of the most appropriate material. Pure tricalcium silicate is ideal for use as substitute for dentine, providing protection and vitality to the pulp.



Authors: C.D.E.E Alberto Segovia Ramírez

1996-1998 Master in Endodontics from Faculty of Dentistry of the U.A.S.L.P. San Luis Potosí, S.L.P.Mexico.

1988-1993 Dental surgeon from Faculty of Dentistry of the U.A.N.L. Monterrey, N.L. México.

Certified as a specialist endodontist by the Mexican Council of Endodontics (no. 369) in May 2004, May 2009 and May 2014 in Mexico City D.F.

Professional and teaching experience:

National Endodontics Research Prize from the Mexican Endodontics Association, 1999.

February-December 1997: Professor of Endodontics and Clinical Adviser in the undergraduate teaching staff at the Faculty of Dentistry of the U.A.S.L.P.

September 1998 - May 2003: Postgraduate Professor of Endodontics in the Dentistry Studies Center of Querétaro.

January 2001 to date: Professor of Endodontics at the Universidad del Valle de México (Querétaro campus).

Guest professor for postgraduate Endodontics at the Universidad San Carlos de Guatemala (Faculty of Dentistry) from 2003 to 2009.

President of the Scientific Committee of the Dentistry Association of Querétaro (2003-2005 and 2005-2007).

Private practice in Endodontics, from 1995 to date, in Monterrey, San Luis Potosí and Nuevo Laredo, and since 1998 to date in Querétaro.

Deicy Álvarez Rodríguez

Dental Surgeon, Universidad Valle de México Campus Querétaro.

Speciality in Restorative Dentistry at the Centro de Estudios Odontológicos, Querétaro.

References

- 1. Gómez de Ferraris. A. campos Muñoz histología, embriología e ingeniería tisular buco dental. En complejo dentino-pulpar II: dentina, (capítulo 11). 3 Edición. Editorial medica panamericana. 2009: pp. 255-270.
- 2. Kenneth J. Anusavice Phillips la ciencia de los materiales dentales. In; dental cements.11 Edición. Elsevier. España: 2004: pp. 471-484.
- 3. Operatoria dental arte y ciencia. Cliford M. Stundervant Theodore M. Roberson Harald O. In: materiales odontológicos; cementos y bases (capítulo 6).3°Edición. Editorial Mosby. España.1996: pp. 236-241.
- 4. Abate P. protección dentino-pulpar. En Basrani E, Cañete M, Blank A. Endodoncia integrada. Colombia. Actualidades Medico Odontológicas, 1999: pp. 191-208.
- 5. Brännstrom M, Nordenvall KJ. Bacterial penetration, pulpal reaction and the inner surface of concise enamel bond. Composite filling in etched and unetched cavites. Journal of dental research 1978: pp. 3-10
- 6. Geddes I. protección dentinopulpar en Mooney B. Operatoria dental. 3ra Edición. Buenos Aires. Editorial medico panamericana, 1999: pp. 692-719
- 7. Camejo M. González O. Solorzano A. Balda R. Protección dentino pulpar. Acta odontológica Venezolana. ISSN: 0001-6365.Vol.37 N°3:1999.
- 8. Pereira JC, Esteves T, Costa L, Cestari T, Ribeiro M, Pagani M. Recubrimiento pulpar directo e indirecto: mantenimiento de la vitalidad pulpar. Acta Odontológica Venezolana. 2011: p. 15.
- 9. Nagaraja Upadhya P. Kishore G. Glass ionomer cement The different generations. Journal. Vol. 18 (2), 2005.
- 10. Hildago R. Mendez M. Ionómero de vidrios convencionales como base en la técnica restauradora de sandwich cerrado: su optimización mediante la técnica de acondicionamiento acido simultáneo y selectivo. Acta odontológica venezolana. ISSN 0001-6365:Vol.47; N°4 Caracas. 2009.
- 11. Priyalakshmi.S¹, Manish Ranjan². Review on Biodentine™- A Bioactive Dentin Substitute. IOSR Journal of Dental and Medical Sciences (IOSR-JDMS) e-ISSN: 2279-0853, p-ISSN: 2279-0861.Volume 13, Issue 1 Ver. III (Jan. 2014), pp. 13-17
- 12. Cedillo J.1, Espinosa R. 2, Curiel R. 2, Huerta A. Nuevo sustituto bioactivo de la dentina tricalcium silicate purificado. Revista RODIB Volumen II. Número 2. 2013; 1-12.
- 13. Laborde J, Giani A, Cedrés C. Una nueva alternativa biocompatible: biodentine™. A new biocompatible alternative: biodentine™. Actas odontológicas-facultad de Odontología de la Universidad católica de Uruguay. Volumen XI/número 1/2014; 11-16.
- 14. Mori G, Teixera L, De olivera D, Jacomini L, Da silva S. Biocompatibility evaluation of biodentine[™] in subcutaneous tissue of rats. JOE. Vol 40, No. 9, 2014.



Louise Vincer - BDS, MFDS.

Introduction

This case describes a novel indirect pulp capping approach to managing gross caries in a healthy tooth. The importance of a thorough clinical examination, including a CBCT scan, is explained. The rationale for single visit use of Biodentine™ is described.

The function of a vital dentine-pulp complex includes dentinogenesis throughout life and in response to injury. The pulp also contains circulating immune cells designed to confront bacterial challenges, and proprioceptive mechanisms to protect against excessive occlusal loading.(1) Indirect pulp capping is indicated on a tooth with a carious lesion in close proximity to the pulp either being asymptomatic or showing signs and/or symptoms of reversible pulpitis. The tooth may be restored permanently, or reentered 6 months later (stepwise excavation).(2) Indirect pulp capping has been shown to have a better prognosis than direct pulp capping.(3) There is good evidence to show that, at the advancing front of a carious lesion, considerable

demineralization of dentine occurs prior to bacterial invasion.(4) A clinical study by Mertz-Fairhurst et al demonstrated that partial caries removal and sealing with composite resulted in the arrest of carious lesions without any signs or symptoms of pulpitis after 10 years.(5) Partial caries removal is the preferred treatment as long as the restoration may be well sealed on a caries-free enamel-dentine junction.(6) From the available literature it seems that a single visit procedure is preferred not only for patient convenience, but also because it has a comparable outcome to complete caries excavation.(3,5,6)

Until recently, calcium hydroxide has been the gold standard pulp capping material for vital pulp therapy.3 However, it is subject to degradation over time and does not adhere well to dentine, and also has tunnel defects forming within the hard tissue barrier.(7) These properties could allow microleakage and failure of the vital pulp therapy. The introduction of bioactive calcium silicate materials such as mineral trioxide aggregate (MTA, Angelus Soluções Odontológicas, Londrina, Brazil) and Biodentine[™] (Septodont, Saint Maur des Fosses, France) offer advantages over calcium hydroxide. Biodentine[™] has been shown to induce mineralization within the dentine8 and is able to induce hard tissue

barrier formation.(9) It is also suitable as a longterm provisional direct posterior restoration.(10) This case report details the use of Biodentine™ as an indirect pulp capping material with partial caries removal performed in a single visit.

Clinical Case

A fit and healthy 24-year-old male presented for a routine dental check-up complaining of a "chipped" lower left molar tooth. He had no other symptoms. The patient had a Class 3 skeletal base and edge-to-edge incisal relationship. This resulted in no anterior guidance, and the LL8 being in occlusion with the opposing UL7.

Clinical examination revealed a cavitated carious lesion associated with the LL8 (Fig. 1a, 1b). There were no signs of endodontic or periodontal disease associated with the lower left molar teeth. All the lower left molar teeth responded positively to thermal (EndoFrost Roeko, Coltène Whaledent, West Sussex, UK) and electric (Vitality scanner, SybronEndo, Peterborough, UK) pulp testing.

A periapical radiograph revealed extensive distal and occlusal caries in very close proximity to the root canal system of the LL8 (*Fig. 2a*). A diagnosis of gross caries was made for LL8. After discussing the various treatment options, the patient consented to a single visit indirect pulp capping procedure. A Cone Beam CT (CBCT) scan confirmed that there were no signs of periapical pathology associated with the LL8 (*Fig. 2b, 2c*).



Fig. 1a



Fig. 1b



Fig. 2a



Fig. 2b



Fig. 2c

The tooth was anaesthetized, and rubber dam applied. The dento-enamel junction was cleared of all caries, leaving stained affected dentine over the axial wall of the pulp chamber.

A 3 mm layer of Biodentine[™] was applied over the affected layer of dentine, followed by glass ionomer cement (Fuji IX, GC Corporation, Tokyo, Japan). The tooth was then permanently restored with a direct composite resin restoration Filtek Supreme XTE (3M ESPE, St Paul, USA) (Fig. 3 a–3d). The patient was reviewed one year later. The patient reported that the tooth was asymptomatic. Clinically and radiographically there were no signs of endodontic disease associated with LL8, and it responded positively to vitality testing (Fig. 4a, 4b).



Fig. 3a



Fig. 3c



Fig. 4a

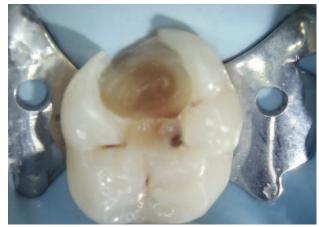


Fig. 3b



Fig. 3d



Fig. 4b

Discussion

To the authors' knowledge, this is the first case report of a single visit Biodentine™ indirect pulp cap. This is a promising technique with many advantages. Performing the treatment in a single session by sealing the affected dentine is less traumatic to the pulp, and is also more efficient for both the patient and practitioner. A two-stage indirect pulp cap requires re-accessing the tooth, multiple appointments and potentially further mechanical trauma to the pulp.

The use of CBCT has been shown to be effective in the diagnosis and management of endodontic problems.11,12 CBCT has a higher sensitivity for detection of early periapical change13 and periapical lesion detection than radiographs.13,14 This is because periapical lesions are not visible on radiographs unless they perforate the junction between the cancellous bone and cortical plate, ie the cortical plate acts as anatomical noise masking the changes in the underlying cancellous bone.(15)

In this case a pre-operative CBCT scan was taken to confirm that there was no periapical radiolucency associated with the LL8. A recent prospective clinical trial study identified that the presence of a pre-operative lesion which was only detectable on the CBCT scan, and not the periapical radiograph, had a negative effect on the outcome of indirect pulp capping.(15,16) Maintaining pulp vitality helps to improve the long-term prognosis of a tooth. It is well established that endodontic treatment mechanically and bio-chemically weakens tooth structure. It has been shown, using micro CT, that access cavity preparation and caries removal can remove up to 20% volume of tooth structure.(17,18) Sodium hypochlorite has been shown to reduce elasticity and flexural strength of dentine.(19) Following root canal treatment, a cuspal coverage restoration is required for posterior teeth in order to reduce the likelihood of a fracture due to the loss of proprioception and weakening of the tooth structure.(1,20) Ultimately, the strength of a tooth is proportional to the bulk of remaining dentine (21) and performing an indirect pulp cap in this case has preserved tooth structure and maintained vitality of the pulp and longevity of the tooth.

Authors:

Shanon Patel

BDS, MSc, MClinDent, MRD, PhD, FDS, FHEA, King's College London and 45 Wimpole Street, London W1G 8SB (shanonpatel@googlemail.com).

Louise Vincer

BDS, MFDS, Postgraduate Endodontic Unit, King's College Dental Institute at Guy's King's and St Thomas' Hospital, London, UK.

References

- Randow K, Glantz PO. On cantilever loading of vital and non vital teeth. An experimental clinical study. Acta Odontol Scand 1986; 44: 271–277.
- European Society of Endodontology. Quality guidelines for endodontic treatment: consensus report of the European Society of Endodontology. Int Endod J 2006; 39: 921–930.
- Hilton TJ. Keys to clinical success with pulp capping: a review of the literature. Oper Dent 2009; 34: 615–625.
- Shovelton DS. A study of deep carious dentine. Int Dent J 1968; 18: 392–405.
- Mertz-Fairhurst EJ, Curtis JW, Ergle JW, Rueggeberg FA, Adair SM. Ultraconservative and cariostatic sealed restorations: results at year 10. J Am Dent Assoc 1998; 129: 55–66.
- Ricketts DN, Kidd EA, Innes N, Clarkson J. Complete or ultraconservative removal of decayed tissue in unfilled teeth. Cochrane Database Syst Rev 2006; 19: CD003808.
- Nair PN, Duncan HF, Pitt Ford TR, Luder HU. Histological, ultrastructural and quantitative investigations on the response of healthy human pulps to experimental capping with Mineral Trioxide Aggregate: a randomized controlled trial. Int Endod J 2008; 41: 128–150.
- Watson TF, Atmeh AR, Sajini S, Cook RJ, Festy F. Present and future of glass ionomers and calcium silicate cements as bioactive materials in dentistry: Biophotonics-based interfacial analyses in health and disease. Dent Mater 2014; 30: 50–61.
- Nowicka A, Lipski M, Parafiniuk M, Sporniak-Tutak K, Lichota D, Kosierkiewicz A, Kaczamarek W, Buczkowska-Radlinska J. Response of human dental pulp capped with biodentine and mineral trioxide aggregate. J Endod 2013; 39: 743–747.
- Koubi G, Colon P, Franquin JC, Hartmann A, Gilles R, Faure MO, Lambert G. Clinical evaluation of the performance and safety of a new dentine substitute, Biodentine, in the restoration of posterior teeth a prospective study. Clin Oral Invest 2013; 17: 243–249.
- Patel S. The use of cone beam computed tomography in the conservative management of dens invaginatus: a case report. Int Endod J 2010; 43: 707–713.
- Patel S, Brady E, Wilson R, Brown J, Mannocci F. The detection of vertical root fractures in root filled teeth with periapical radiographs and CBCT scans. Int Endod J 2013; 46: 1140–1152.
- Hashem D, Mannocci M, Patel S, Manoharan A, Brown JE, Watson TF, Banerjee A. Clinical and radiographic assessment of the efficacy of calcium silicate indirect pulp capping: a randomized clinical trial. J Dent Res 2015; 94: 562–568.
- Abella F, Patel S, Duran-Sindreu F, Mercade M, Bueno R, Roig M. Evaluating the periapical status of teeth
 with irreversible pulpitis by using cone beam computed tomography scanning and periapical radiographs.
 J Endod 2012; 38: 1588–1591.
- Kanagasingam S, Mannocci F, Lim CX, Yong CP, Patel S. Diagnostic accuracy of cone beam computed tomography scans and periapical radiography in detecting apical periodontitis using histological findings as a gold standard. Int Endod J (in press).
- Patel S, Wilson R, Dawood A, Foschi F, Mannocci F. The detection of periapical pathosis using digital periapical radiography and cone beam computed tomography Part 2: a 1-year post-treatment follow-up. Int Endod J 2012; 45: 711–723.
- Bender IB, Seltzer S. Roentgenographic and direct observation of experimental lesions in bone II. J Endod 1961; 11: 707–712.
- Ikram OH, Patel S, Sauro S, Mannocci F. Micro-computed tomography of tooth tissue volume changes following endodontic procedures and post space preparation. Int Endod J 2009; 42: 1071–1076.
- Grigoratos D, Knowles J, Ng YL, Gulabivala K. Effect of exposing dentine to sodium hypochlorite and calcium hydroxide on its flexural strength and elastic modulus. Int Endod J 2001; 34: 113–119.
- Salehrabi R, Rotstein I. Endodontic treatment outcomes in a large patient population in the USA: an epidemiological study. J Endod 2004; 30: 846–850.
- Hock D. Impact resistance of post and cores. Thesis, University of Michigan, 1976.



The Dental Pharmaceutical Company