Temposcell

Cemento para sellado provisional

Composición

Base: Óxido de zinc, parafina, excipientes
Catalizador: Resina natural, eugenol, excipientes

Propiedades

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Temporary luting cement

Composition

Base: Zinc oxide, paraffin, excipients.
Catalyst: Natural resin, eugenol, excipients.

Properties

TEMPOSCCELL is a temporary luting cement used for permanent prosthesis, crowns, bridges, or splints. The excellent fluidity of TEMPOSCCELL enables the prosthesis to be easily and completely placed. It is strong enough to resist chewing stress. The prosthesis can be easily removed at any time.

TEMPOSCCELL complies with ISO 3107:2004 specifications type I – class I.

Indications

Temporary luting.

Contraindications

Do not use the material if allergies to ingredients or contact allergies are known to exist.

Side effects

In individual case, it is possible that pulp damage may occur in conjunction with the use of materials containing eugenol.

Instructions for use

- Remove the cap by turning 90° anticlockwise and discard it.
- Attach a mixing tip to the syringe. Line up the tip notch and the syringe notch.
- Turn the mixing tip 90° clockwise until it locks.
- The cement is automatically mixed in the tip and can be applied directly.
- At first use, extrude pea-sized amount of material and discard it.
- Apply the mixed cement into the dry temporary restoration.
- Insert the temporary restoration within 1 min of starting mixing; fix it onto the dry prepared tooth, exerting gentle pressure.
- After curing, remove excess cement with a probe or a scaler.
- Carefully remove interdental excess with dental floss.

Leave the used mixing tip on the syringe to serve as a cap until next use.

Technical data

Working time: 1 min
Intraoral curing time: 7 min (at 37°C)

Warnings and precautions for use

- Avoid contact with skin and eyes. In case of accidental contact rinse immediately with plenty of water and consult a doctor if necessary.
- Contains colophony.
- Eugenol-based temporary sealers may affect the setting of composite and resin-based cements, as well as bond agents. Check to ensure that cements containing eugenol are suitable for use.

Storage

Store at 15-25°C.
Do not store material in refrigerator.
Do not use after the expiration date.

Presentation

2 x 11 g syringes + 20 mixing tips
50 mixing tips

For professional dental use only