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Carboxicelulosa de calcio  
Polvo hemostático estéril reabsorbible

## Composición

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**Calcium carboxycellulose**  
**Sterile, resorbable, haemostatic powder**

## Composition

100 % calcium hydrogen salt of oxidised cellulose.

## Properties

RESORCELL Powder is a sterile, resorbable, haemostatic powder, indicated for the checking of capillary bleeding. Its onset of action is immediate and a perfect haemostasis is obtained within 2 minutes of application. On contact with blood, the powder turns red to dark brown.

## Indications

Haemostasis after tooth extraction, scaling, curettage, gingivectomy, ridge levelling, alveolectomy. It can be used in adults as well as in children.

## Contraindications

No contraindications have been reported.

## Interactions

No interactions with other medicinal products and no other forms of interaction are known.

## Directions for use

After opening the bottle and setting up the canula, pour a small quantity of powder in a Dappen dish. Using a scaler, take the required quantity of powder and apply it to the area to be treated.

**Never reuse the powder remaining on the dish.**

## Special warnings and precautions for use

Do not use on infected tissues.

This preparation contains no antiseptic agent. Its use does not exempt you from cleaning and disinfecting the area to be treated.

After opening of the bottle and, a fortiori after first use, the sterility of the remaining powder is no longer guaranteed, the same is true when the package which guarantees sterility of the powder has been damaged.

RESORCELL Powder is not suitable in case of arterial or venous haemorrhage, where the powder would be washed away by the blood flow.

## Storage

Protect from light and moisture, do not store above 25°C.

## Expiry date

Do not use the powder after the expiry date stated on the packaging.

## Presentation

2 g plastic bottle of powder with a plastic canula.

They are both packaged in a peelable blister, sterilised by gamma rays.

*For professional use only in the practice of dentistry.*