FR

**COMPOSITION**

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EN

**COMPOSITION**

Zinc oxide, Zinc Sulphate, Excipients

**INDICATIONS**

Septo-pack is a malleable self-hardening paste.

*In periodontology*: Septo-pack is a periodontal dressing for the protection of the wounds after deep curettage or periodontal surgery.

It can also be used as a neutral medium to drugs to keep them in place at the gingival, tooth or alveolar level.

*In operative dentistry*: Septo-pack is also a temporary filling material for inlay or onlay cavities or a gingival retraction material for cervical cavities.

**CONTRAINDICATIONS**

Septo-pack is contraindicated on patients with a known allergy to one of the components of the product.

**INSTRUCTIONS FOR USE**

Take a small quantity of product from the jar with a dry, sterile spatula and do not wet the material throughout manipulation. Tightly close the jar after use.

*In periodontology*: After forming the material into a small roll of the required length and thickness, apply Septo-pack onto the area to be covered. Press the material into the interproximal spaces to anchor it and to limit the gingival compression. Ensure that the dressing does not extend on the occlusal surfaces and beyond the immediate area to be protected. The material should not extend beyond the muco-gingival margin to avoid mechanical irritation of the alveolar mucosa.

Working time in the mouth is only 2 or 3 minutes following application. Hardening is complete in half an hour.

If a prolonged protection of the wound area is needed, it is better to renew the material after 7 days. Septo-pack should not be renewed more than once.

*In operative dentistry*: A thin layer of non-greasy lubricant should be applied onto the cavity walls to facilitate the subsequent removal of the temporary restoration. Delicately dry the cavity with air. Apply the required quantity of Septo-pack in the cavity with a spatula.

To remove this temporary filling in one piece, use a scaler or any other instrument with a sharp point.

Eliminate any material left before permanently sealing the inlay or onlay.

**PRECAUTIONS FOR USE**

Xi - Irritating to eyes.

N - Dangerous for the environment

Very toxic to aquatic organisms, may cause long-term adverse effects in the aquatic environment. Avoid release to the environment.

Use appropriate container to avoid environmental contamination.

**PRESENTATION**

Jar containing 60 g of paste.

**STORAGE**

Do not store above 25°C and protect from moisture.

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