PRESENTATION

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INSTRUCTIONS FOR USE

Step 1: Tear the pouch in the middle as shown.

Step 2: Allow the injection device to fall onto the table, holding the packet in both hands (avoid handling the device so that the product's sterility is not altered).

Step 3: Grip the barrel firmly and fully insert the Cartridge into the open end of the device.

Step 4: Grip the handle plunger and put the thumb behind the finger holder. Introduce the handle tip into the barrel of the device, behind the cartridge.

Step 5: Now slide the sheath protecting the needle backwards, towards the handle until it CLICKS (the click is made as the sheath hits the handle and locks the unit together).

Warning: Failure to retract the sheath fully until you hear the click may result in the device disassembling during use.

Step 6: All movements now are away from the needle.

Remove the needle cap and discard. The device is now ready to use. You should always ensure a minimum of 0.25 ml to 0.35 ml of solution has been previously expelled from the cartridge (providing space).

Self Aspiration: At the base of the cartridge barrel of the injection device you will see there is a small protuberance. It appears as a plastic tip holding the centred back end of the needle that penetrates the diaphragm of the cartridge when inserted. At the beginning of the injection, the diaphragm presses against the protuberance, and aspiration occurs as the diaphragm moves back away from the protuberance when released.

Aspiration: The second form of aspiration can be obtained by the pliable O-Ring that is situated at the end of the handle shaft (piston) this creates a vacuum when the handle is pulled back by the practitioner. The cartridge bung follows the piston providing active aspiration. It is best observed when a minimum of 0.25 ml to 0.35 ml of solution has been previously expelled from the cartridge (providing space).

Warning: Should the device be inadvertently fully locked in position 2, no attempt whatsoever should be made to unlock it. Use a new injection device.

Step 7: After you have finished the injection, remove your fingers from the O-Ring Handle and grip the finger holder, slide the protective sheath down the barrel of the syringe until you reach the first holding position, then grip the barrel of the syringe and pull the protective sheath down further to the final locking position.

Step 8: If you wish to remove the Cartridge from the Syringe then, pull the Handle plunger back fully (A), and then peel off the Handle (B). Then pull the finger grip back up the plunger until it comes to the O-Ring Handle then (C) push the plunger back into the Cartridge and remove the Cartridge.

Warning: Should the device be inadvertently fully locked in position 2, no attempt whatsoever should be made to unlock it. Use a new injection device.

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