FR

### COMPOSITION

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EN

### COMPOSITION

Hydroalcoholic solution (ethanol 55 %), quaternary ammonium propionate, fragrance.

### INDICATION

Fast-acting disinfectant of precleaned, alcohol resistant medical devices.

### INSTRUCTIONS FOR USE

Ready-to-use solution.

For use on precleaned medical devices.

Spray onto areas to be treated until they are completely moistened.

Let stand for 30 seconds.

### Note :

Check that surfaces to be treated are compatible with alcohol.

Electrical appliance should be turned off.

### WARNINGS AND PRECAUTIONS FOR USE

Flammable.

Keep out of the reach of children. Keep away from sources of ignition — No smoking.

Use only in well-ventilated areas. Do not breathe the sprayed product. Wear suitable gloves.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

If swallowed, seek medical advice immediately and show this container or label.

Do not reuse the packaging. Avoid release to the environment. This material and its container must be disposed of as hazardous waste.

### MICROBIOLOGICAL PROPERTIES

Bactericidal in 30s: EN 1040, EN 1276, EN 13727, EN 14561.

Active against MRSA in 30s : EN 14561.

Tuberculocidal in 30s : EN 14348.

Yeasticidal in 30s : EN 1275, EN 13624.

Active in 30s against HIV-1, BVDV (HCV-like virus), PRV (HBV-like virus), Rotavirus, Herpesvirus, Adenovirus, Coronavirus and Influenza Virus [H1N1].

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