FR

COMPOSITION

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EN

COMPOSITION

Water-soluble lubricant, edetic acid, carbamide peroxide.

PROPERTIES

- lubricates the root-canal and facilitates endodontic instrument penetration,
- makes shaping more efficient,
- due to its effervescent properties makes cleaning of the root-canal easier.

INDICATION

Facilitates root-canal preparation.

CONTRAINDICATION

Allergy to one of the ingredients.

INSTRUCTIONS FOR USE

1. Coat the instrument to be used with the gel for lubrication;
2. Shape to the desired number;
3. Irrigate with a sodium hypochlorite solution.

OPERATIVE TECHNIQUE**1. Pulp chamber:**

- Isolate the tooth to be treated, using a rubber dam;
- Open up the pulp chamber in the usual manner;
- Fit the applicator tip to the CANAL PLUS syringe;
- Place CANAL PLUS;
- Irrigate with a sodium hypochlorite solution to remove pulp debris and clear the entrance to the root-canal(s);
- Remove the liquid from the chamber by aspiration.

2. Root-canal:

The way CANAL PLUS is used will vary, depending on the anatomical features of the root-canal(s) to be prepared. Incorrect direction of reaming should be avoided in any instance. For preparation of wide and relatively straight root-canals, CANAL PLUS can be used starting with the first instrument number. For narrow, calcified and/or irregular shaped canals it may be necessary to assess the shape and working length and to start reaming up to a rather high number before applying CANAL PLUS. The instruments should be pre-curved and adjusted (stopped) to facilitate the work.

Then proceed by coating the instruments with CANAL PLUS, working in the usual way; then rinse with hypochlorite, dry and fill the canal as normal.

Notice

Avoid using CANAL PLUS right up to the last numbered reamer. Although it is highly water soluble, always make sure no trace of CANAL PLUS remains in the canal.

CANAL PLUS should not be left in the tooth between visits.

SPECIAL WARNINGS AND PRECAUTIONS FOR USE

Xn – Harmful (contains hydrogen peroxide)

Harmful if swallowed. Risk of serious damage to eyes.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

Wear eye/face protection.

This material and its container must be disposed of as hazardous waste.

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